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
ایمیل شرکت: Info@tikadentplus.ir

Instruction to use Tika-BurTM CTT

Read the instructions carefully before using.

Schematic image to illustrate the main parts

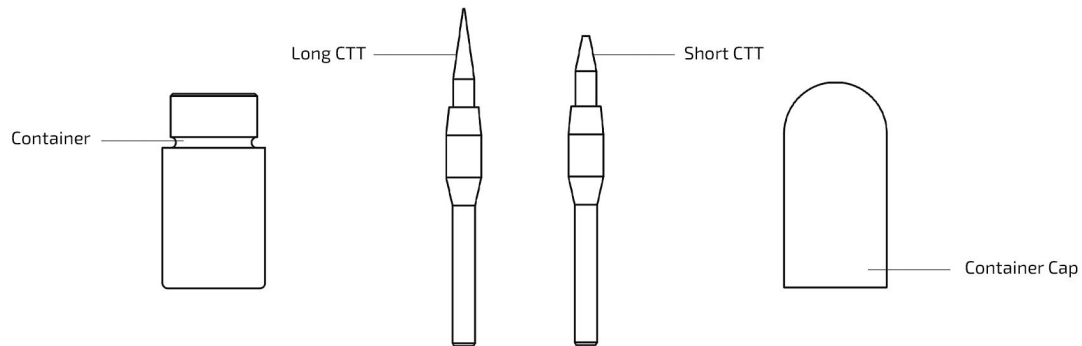[en]

Product Description:

Tika-Bur™ CTT Ceramic Bur is a specialized rotary instrument designed for soft tissue cutting in dentistry. This tool can be attached to a high-speed handpiece (turbine) and used for gingival tissue cutting. This instruction manual has been prepared to ensure the proper use of the ceramic bur.

Applications:

Some of the applications of the ceramic bur include:

- Gingivectomy
- Operculectomy
- Esthetic crown lengthening
- Uncovering intraosseous implants excising granulation tissue
- Exposing fractured teeth and unerupted teeth
- Contouring of inflamed gingival soft tissue.
- Exposing deep cervical caries or external resorption.

Sterilization:

- Tika-Bur™ CTT is delivered in non-sterile packaging. Therefore, must be sterilized before use each time.
- Tika-Bur™ CTT must be autoclaved in standard autoclave Equipment at 134°C for 3 minutes.
- It is recommended to sterilize the bur inside the metal box in the autoclave.

Usage tips:

- Place the Tika-Bur™ CTT in air turbine handpiece, and make sure it's properly placed and fixed.
- Tika-Bur™ CTT operates at a speed of 300,000 to 500,000 rpm.
- Make sure water supply of the air turbine handpiece is turned off, and air turbine handpiece is used with non-water coolant. Otherwise the necessary/proper heat to remove tissue and coagulation of blood vessels is not generated, which results in bleeding.
- Avoid contact of Tika-Bur™ CTT with bone, teeth, dental restorations and metallic components of implants during surgery, as this can cause color change or break the ceramic head of the bur.
- In case of increasing gingival groove depth, pay attention to preserve the structure of marginal epithelium and the level of keratinized gingiva.
- Tika-Bur™ CTT can be used several times as long as its ceramic head is undamaged.

Usage instruction:

- Before using the Tika-Bur™ CTT, ensure that the area is dry and free of any liquid and blood.
- For optimal results, Tika-Bur™ CTT should be moved gently and delicately within soft tissue at the angle of 45-30 degrees. Avoid rapid and deep movements.

After-Sales Service

The product is covered by a 1-year warranty. This warranty does not cover damages to the device caused by impact, washing with chemicals, use and maintenance of the device outside the provided instructions.

Manufactured by Pooyan Teb Tika Co. Ltd.

WWW.TIKADENTPLUS.IR

Info@tikadentplus.ir

No. 404, Andisheh 1 Building,
Isfahan Science & Technology Town, Isfahan, Iran