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
Instruction to use Tika-Prep™ Warmer-NL

Read the instructions carefully before using.

Schematic image to illustrate the main parts

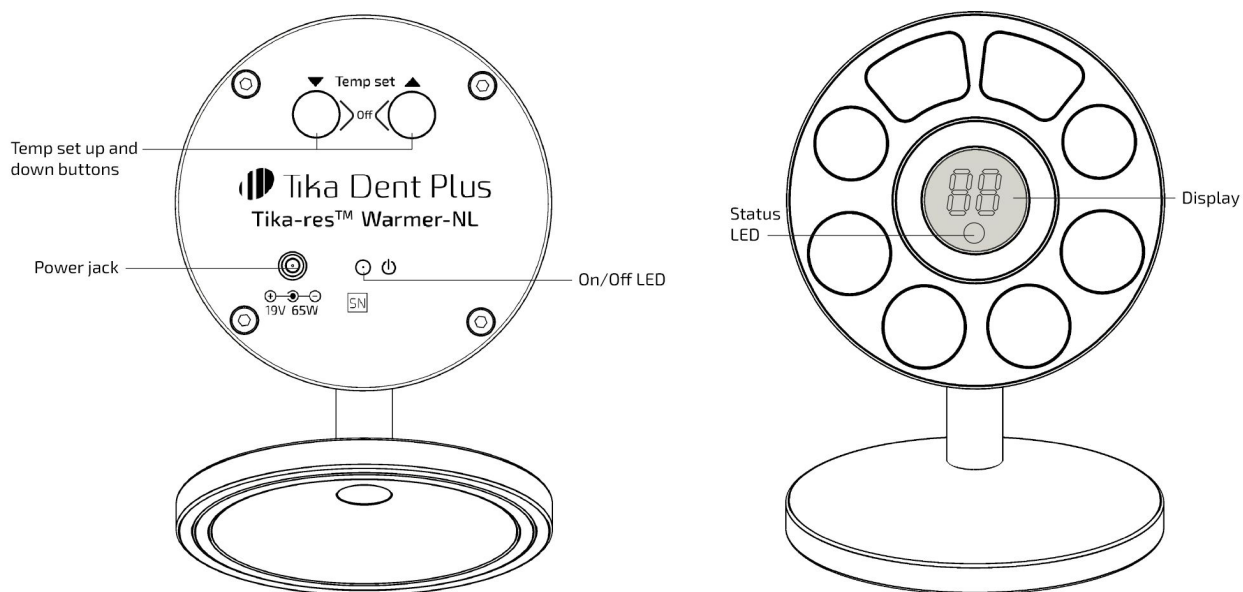[en]

Product Description:

Tika-Res™ Warmer-NL is a professional device designed for heating dental materials such as various types of composite, local anesthetics, and sodium hypochlorite. With precise temperature adjustment ranging from 35°C to 70°C, this device allows users to heat their materials in different sizes at their desired temperature.

Key features of this product include a user-friendly design, a digital temperature display, and a temperature status indicator light.

Applications:

Tika-Res™ Warmer-NL is specifically designed to provide heat to various materials such as:

- Dental Composites
- Local Anaesthetic (LA)
- Sodium Hypochlorite

Precaution:

- Exercise caution as the unit may become hot to the touch when in operation at high temperatures, particularly the metallic surfaces.

Setup

- To turn on the Tika-Res™ Warmer-NL, connect the adapter plug to the power jack. Once connected to a power source, the display screen and the red LED light at the back of the device will turn on.
- Use the temperature set up and down buttons to adjust the desired temperature.
- The blue status LED light indicates that the device is gradually heating up.
- Depending on the set temperature, the time required to reach the desired temperature is approximately 5 to 10 minutes.
- The green status LED light indicates that the device has reached the set temperature.
- If the device temperature exceeds the user-set temperature, the red status LED light will turn on, and the device will gradually cool down.
- To turn off the device, press and hold both temperature adjustment buttons simultaneously for one second.
- If the device remains on for more than 4 hours without any changes in settings, it will automatically turn off.
- It is recommended to manually turn off the device if it is not in use for more than one hour.

Parameters:

- Accuracy of temperature controlling: 0.1 degrees
- Display accuracy: 1 degree
- Temperature range: 35-70 degrees Celsius

Cleaning/Disinfection Instruction:

- It is possible to clean Tika Warmer by wiping them with a wet alcohol wipe.
- Avoid submerging any parts in liquid.
- Avoid autoclaving any parts of the unit.

After-Sales Service:

The device is covered by a 1-year warranty on materials and construction. This warranty does not cover the adapter and damages to the device caused by impact, power fluctuations, liquid intrusion, fire, repairs and servicing by unauthorized persons, use and maintenance of the device outside the provided instructions, and any unforeseen incidents.

Technical Data:

- Voltage: 19 VDC
- Power: 65 W
- Temperature range:
Operating: 0°C – 40°C
Storage: -20°C – 60°C
Humidity: 10 – 95%

Manufactured by Pooyan Teb Tika Co. Ltd.

WWW.TIKADENTPLUS.IR

Info@tikadentplus.ir

No. 404, Andisheh 1 Building,
Isfahan Science & Technology Town, Isfahan, Iran